

AXOLTM Procto Cream

A LUBRICATING, ANTI-INFLAMMATORY, PROTECTIVE and REFRESHING ACTION IN PRESENCE OF:

**- INTERNAL / EXTERNAL HAEMORRHOIDS
- PROCTITIS**



CE Class 1
Directive 93/42/EEC

products may vary

PRODUCT CLASSIFICATION:
European Medical Device.

ACTIVE INGREDIENTS:

- ◆ Escin
- ◆ Menthol
- ◆ Vitis vinifera
- ◆ Eureka

HOW TO USE: the cream is for external and internal use.

For external use: put a pinch of cream in the perianal area.

For internal use: screw the cannula on the tube and squeeze the cream into the endorectal area. Use the product after defecation. The treatment can be repeated 2/3 times a day. After use, unscrew the cannula and clear it with soap under hot water.

PACKAGING: Aluminum tube of 40 ml inserted in a fold box with leaflet.

ESCIN:

It acts on the capillaries of the hemorrhoidal veins, by stimulating the circulation of blood. It helps to improve the blood-flow, reducing the oedema and helping hemorrhoids to retract.

MENTHOL:

Thanks to its well-known local anesthetic properties, it is very useful against pain caused by hemorrhoids.

VITIS VINIFERA:

Plays a protective action at the level of the microcirculation by decreasing the permeability of venous capillaries and by increasing their resistance also in presence of recurrent proctitis.

EUREKO HC:

This polymer contains hydrogen peroxide and it frees oxygen. As a result, a disinfectant action and a healing effect on the local tissues are obtained on the acute hemorrhoidal crisis